

Casuarina Aquatic and Leisure Centre Community Room

Hearing Assistance Devices Usage Guide

The neckloops provided connect the audio output from the IR Receiver unit into a hearing aid equipped with a T-coil. Headsets have also been provided for those without hearing aids.

TO USE WITH A T-COIL HEARING AID

- Place the IR Receiver unit's cloth lanyard and the neckloop around your neck and slide the neckloop's cord lock up to a comfortable position
(Manufacturer's note: cord lock adjustment will slightly affect the induction field strength)
- Plug the neckloop into the headphone jack on the IR Receiver unit
- Turn on the unit and set the volume to 3
- Turn the switch on your hearing aid to the T position and adjust the volume for comfortable listening
- If the sound is weak, increase the volume on the IR Receiver unit



TO USE WITHOUT A T-COIL HEARING AID

- Place the IR Receiver unit's cloth lanyard around your neck
- Plug the headset provided into the headphone jack on the IR Receiver unit
- Turn on the unit and set the volume to 2
- Place headset over your ears
- If the sound is weak, increase the volume on the IR Receiver unit



NOTES

Turn OFF IR Receiver unit and return to its charger when finished using

Report any malfunctioning equipment or any other issues in the room to City of Darwin at darwin@darwin.nt.gov.au

Turn OFF lights and air-conditioning before leaving the Community Room